

Inogen One G3

Backpack - Instructions



1. Open top half of backpack and remove battery spacer insert if you are using a double battery (16 cell). Do not remove the insert if you are using a single battery (8 cell).



2. Insert the Inogen One G3 concentrator in place by securely attaching the velcro strap around the concentrator. To verify you have inserted the concentrator into the backpack in the proper direction, you will be able to see the top display screen through the plastic window of the backpack.



3. Zip up the top portion of the backpack.



4. Insert cannula through the cut out of the top of the backpack and attach the tubing securely onto the cannula barb.

Inogen One G3

Backpack - Instructions




5. Verify vents are not blocked. There is a cut out on the top of the backpack for connection to external power.



6. Lower bottom pocket for your extra accessories.



7. Buckle and pull straps to adjust for proper fit. (Optional strap can be removed).

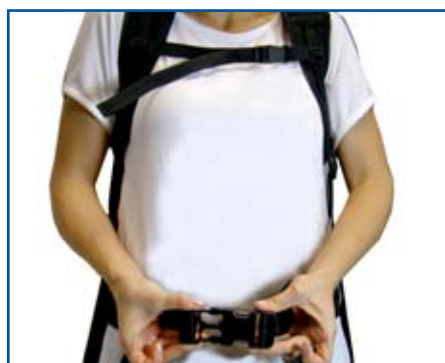

8. Buckle and pull straps to adjust for proper fit. (Optional strap can be removed).