

5 Quick Steps to Recharge Your Battery.

INOGEN ONE External Battery Charger

Your Equipment Provider:

Your Physician:



120 Cremona Drive, Suite B
Goleta, CA 93117
1-877-4-INOGEN (Toll Free)
1-877-446-6436 (Toll Free)
www.inogen.net
www.OxygeNation.com

96-00025-00-02A



INOGEN ONE

External Battery Charger

Your new Inogen One External Battery Charger (BA-103) will charge your Inogen One Battery. It takes approximately 3 hours to charge a fully depleted battery. Note: the battery will also recharge in the Inogen One when powered by the AC Power Supply.



Plug the External Battery Charger AC power supply cord into the nearest electrical outlet.



Plug the External Battery Charger AC power supply into the battery charger.



An illuminated red light on the left of the charger indicates the power is on.



Slide your Inogen One Battery onto the charger with the handle of the battery up. The battery will only fit one way, and easily plugs into place as shown.



When the battery is in the correct position, a flashing green light will indicate that the battery is charging. When the green light stops flashing the battery is fully charged.*



*If the red power light is flashing, unplug the unit and complete steps 2-4 again. If a red flashing light continues after repeating these steps, contact your Equipment Provider.