

INOGEN ONE

Satellite Conservor®

The Inogen One® Satellite Conservor® (SC-100) is a great way to achieve independence around the house, office or other central locations where you want up to 100 feet of freedom of movement. Setting up your Inogen One Satellite Conservor is easy by following these seven steps.

Quick Start Instructions:

Before you start ...

Call Inogen Customer Care at 1-877-466-4362 to configure your Inogen One for use with your Satellite Conservor. The process should only take a few minutes.



Attach belt clip to your Satellite Conservor by sliding into place (if desired).



Attach desired length of the tubing (25 to 100 ft.) to metal nozzle on your Inogen One.



Attach the other end of the tube to the metal nozzle labeled "SUPPLY" on your Satellite Conservor.



Attach nasal cannula to nozzle labeled "CANNULA" on your Satellite Conservor.



Adjust the flow setting on your Inogen One to "Satellite" by pressing the increase flow setting button, marked with a "+" symbol, until "Satellite" is displayed on the screen. **NOTE: You must call Inogen @ 1-877-466-4362 to configure the Inogen One for use with the Satellite Conservor.**



Adjust the flow setting on your Satellite Conservor to the correct flow setting prescribed by your physician.

General Instructions For Use:

Satellite Conservor


If you have a Satellite Conservor call Inogen Customer Care at 1-877-466-4362 to configure the Inogen One for use with the Satellite Conservor.

The Satellite Conservor enables use of the Inogen One by placing the concentrator in one location and connecting additional lengths of tubing.

Place the concentrator in the desired location and power with the AC Power Supply.

Attach the desired length of supply tubing (25 to 100 feet) to the hose nozzle on the Inogen One and to the "SUPPLY" hose nozzle on the Satellite Conservor.

Attach the cannula to the "CANNULA" hose nozzle on the Satellite Conservor.

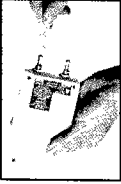

inogen