


Quick Setup Guide *

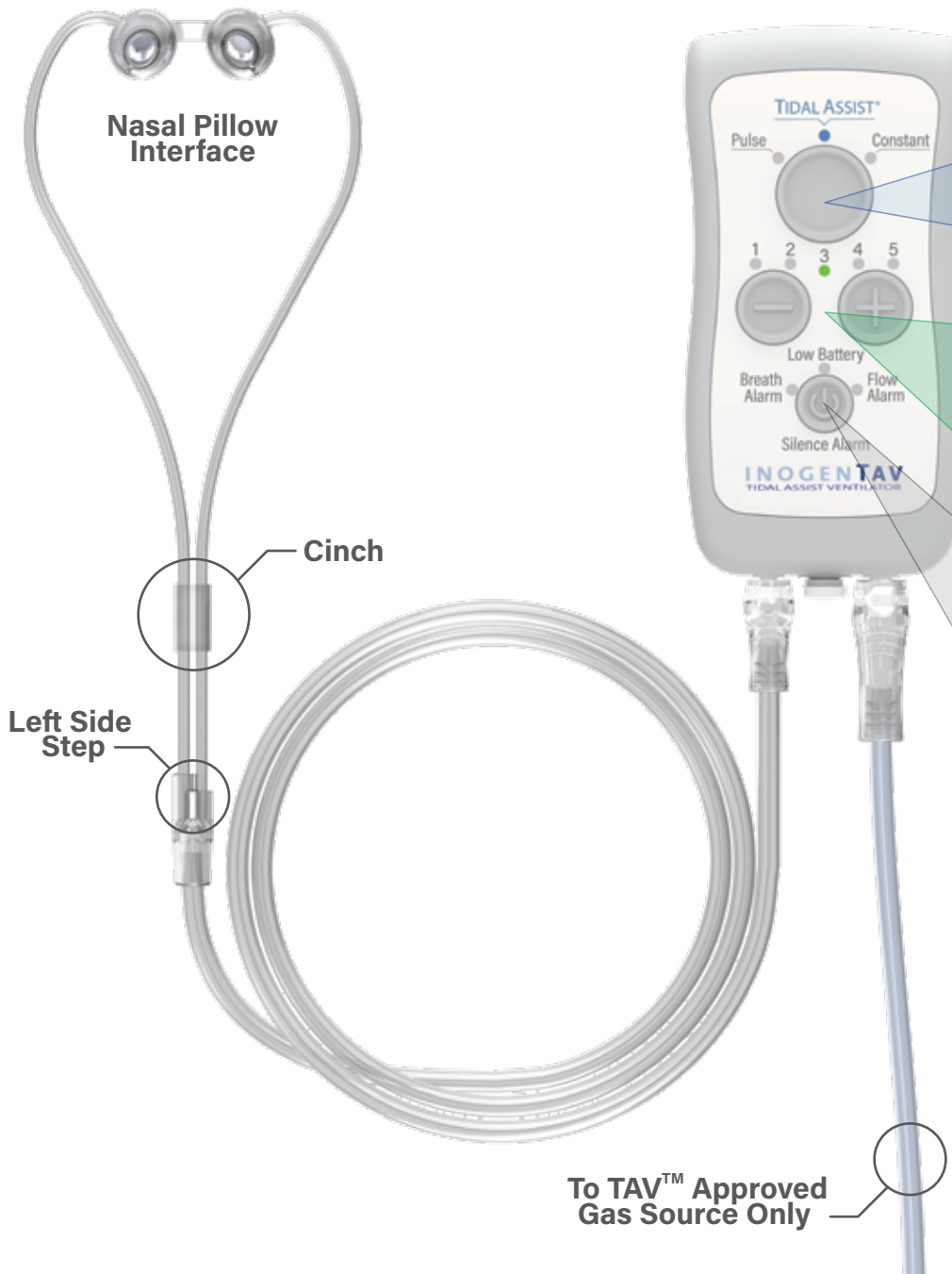

Mode Page 10 13

TIDAL ASSIST™

Pulse Constant

Tidal Assist

Pulse Constant

Level Page 10 13

DECREASE FLOW

INCREASE FLOW


Power / Silence Alarm Page 10 13

Hold 3 seconds
On / Off

Press briefly to
Silence Alarm

* Reference Instructions For Use for additional information
96-09783-00-01 C, TAV QUICK REFERENCE GUIDE

Quick Troubleshooting Guide *

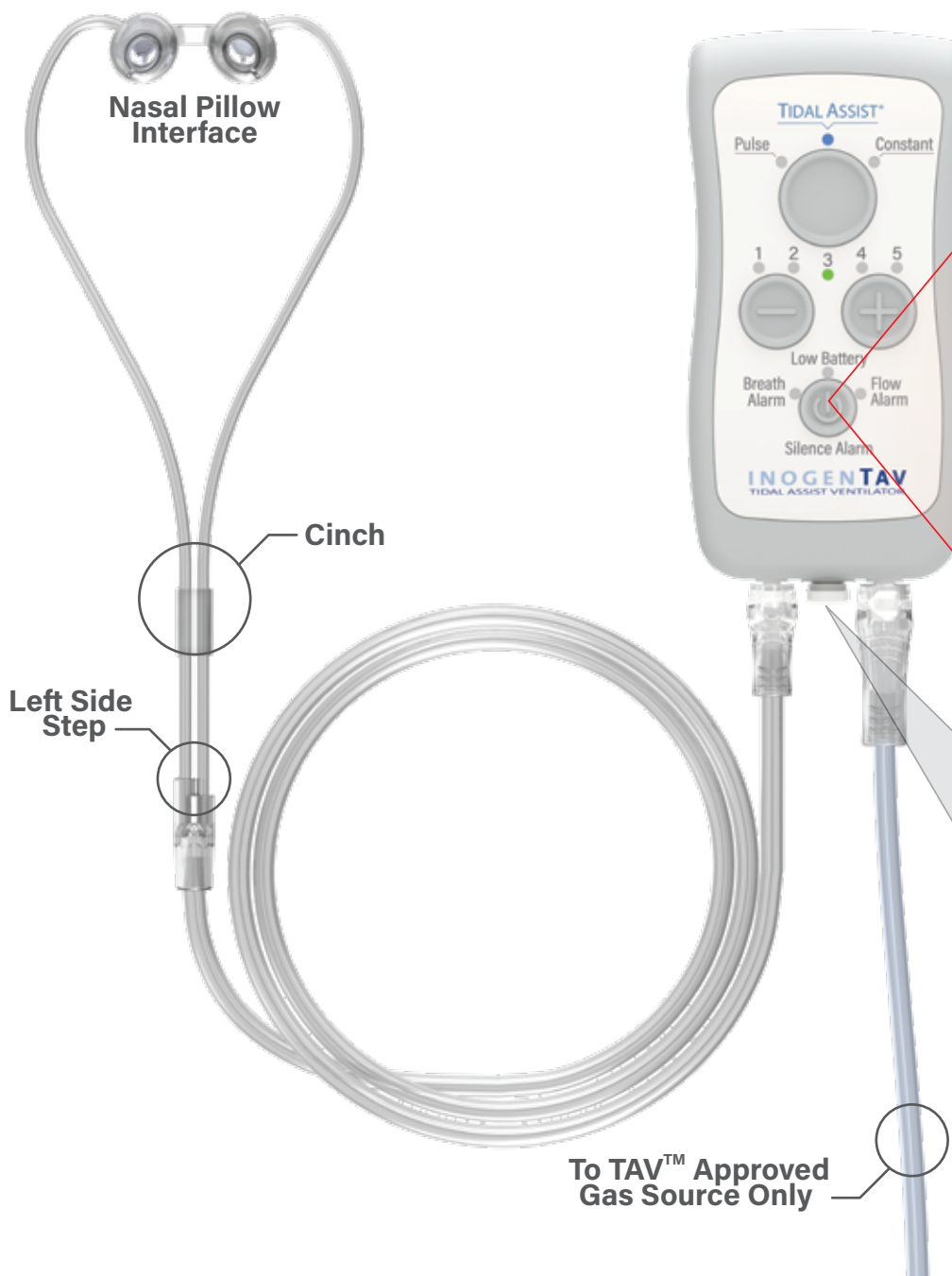

Alarms



1. Check patient
2. Check nasal pillow interface fit
3. Check tubing for kinks and leaks
4. Check all connections
5. Turn Device off and then back on



Battery low - replace battery



1. Check tubing for kinks and leaks
2. Check connections
3. Check gas source
4. Check regulator adaptor
5. Turn Device off and then back on



1. Check Bypass Valve is closed
2. Turn Device off and then back on


Note: if alarms cannot be cleared, turn off device and call customer support. Use Bypass Valve if needed.

Bypass Valve (2LPM constant)

Power off



Hold 3 seconds

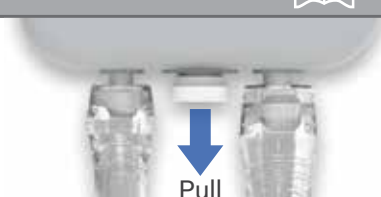

Pull

* Reference Instructions For Use for additional information