inogen connect  
GETTING STARTED GUIDE  
Android  
Apple

# Before You Start

**1 Download App**  
From your smart device, go to the **APP STORE (APPLE)** or **Google Play (ANDROID)**, search for the **Inogen Connect App** and download the software. A download link is also included in your welcome email. Please double click the underlined text in your email from Inogen to download the software.

**2 Plug In Your Inogen One G4**  
Connect the AC power supply cord to the concentrator. Plug the power supply AC cord into an electrical outlet. **Do not power on G4 concentrator.**

inogen connect

# Pairing

**3 Pair Your Mobile Device With Your G4**


- Go to **Settings** on your smart device and **turn on Bluetooth**.
- Slide the button to the right to turn on Bluetooth.
- Next, **turn on Bluetooth on your concentrator** by holding down the minus button until the Bluetooth symbol appears on the concentrator's display.

**4 Open Inogen Connect App**  
The Inogen Connect Code can be found in your confirmation email or provided by your home care provider. Enter your Inogen Connect Code by:

- Manually entering.
- Use your smart device's camera to scan the QR code found on your invoice (must allow access to your camera through the app)


*Pairing is the term used to describe the process of wirelessly connecting two Bluetooth devices for the first time. "Pairing" allows the devices to recognize each other and creates a unique lasting connection between the specific devices.*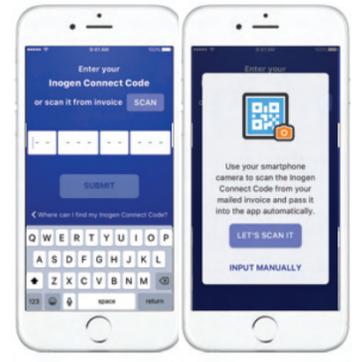


inogen connect

# Pairing

**5 Search for Concentrator**  
Click on "Search for Concentrator" at the bottom of your screen.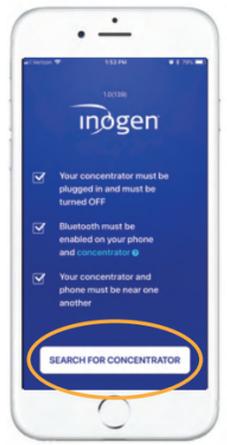

**6 Select Serial #**  
When device is found, select your desired **Concentrator Serial Number** on your smart device.

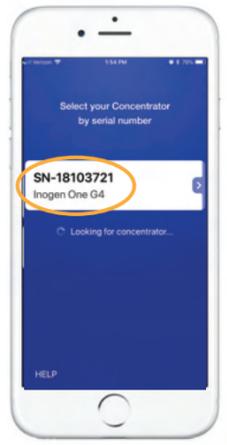**7 Terms & Conditions**  
Read the **Terms and Conditions** and if you choose to accept, select the "I Accept" button at the bottom of your screen. There may be an additional consent statement for some locations. Read the **consent statement** and if you choose to accept, select the "I Accept" button at the bottom of your screen.

**8 Press Bell Button**  
Press and hold the **bell button** on your G4 concentrator to finish pairing.

inogen connect

**9 Wait to Pair**  
Please wait - your concentrator is pairing.

**10 Pairing Complete**  
Your **Inogen One G4** is now paired with your smart device! Now you can turn on your Inogen One G4 and use it normally. The information displayed on your **Inogen Connect** screen will vary depending on your concentrator's current state.

For further information, see the Inogen Connect FAQ section at [www.inogen.com/app/](http://www.inogen.com/app/)

THE PURPOSE OF THE INOGEN MOBILE APP IS TO PROVIDE ADDITIONAL CONVENIENCE TO THE PATIENT. HOWEVER, THE USER INTERFACE PANEL REMAINS THE PRIMARY SOURCE OF INFORMATION TO WHICH THE PATIENT SHOULD REFER.

inogen connect

**Location Information.** When you use our mobile applications, we receive your precise location information. We may also collect the precise location of your device when the app is running in the foreground or background / when the app is closed. We use your location information to monitor the location of our equipment, locate lost or stolen equipment, perform market analysis, and locate nearby service representatives and branch offices. We also infer your more general location information (for example, your IP address may indicate your more general geographic region). We may disclose the information we collect about you (including your location information) with homecare providers involved in your care. The Inogen Connect App is compatible with the following devices: iPhone 5 and later; iPad Air; iPad Air 2; iOS 9 and later; Samsung S5 and later; Nexus 5; Nexus 6; Nexus 6; Nexus 6; Android 6 and later. Compatible device models are subject to change at any time without notice.

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Inogen, Inc.