

Instructions for Enabling the Inogen One for use with the Satellite Conserver

- 1) With the Inogen plugged into AC power and turned OFF, press and hold down both the MODE and the LIGHT buttons at the same time for 6 seconds until the screen says, "Mode select normal".
- 2) Release those buttons, press and release the PLUS button 3 times until it says "Satellite"
- 3) Then press and release the MODE button one time and it will say "Satellite disabled"
- 4) Then press and release the PLUS button one time and it will say "Satellite enabled"
- 5) Then press and release the MODE button one more time and the screen will go dark
- 6) Then the next time you turn on the Inogen, go past flow setting 5 to Satellite. Please refer to the Owner's Manual for instructions on operating the Inogen One and the Satellite Conserver.

This is a one time set up and never needs to be repeated again.